

BATH SAFETY


McKESSON

Transfer Bench

aluminum frame

Provides a total weight bearing capacity of 400 lbs (181 kg).

INSTRUCTIONS FOR USE

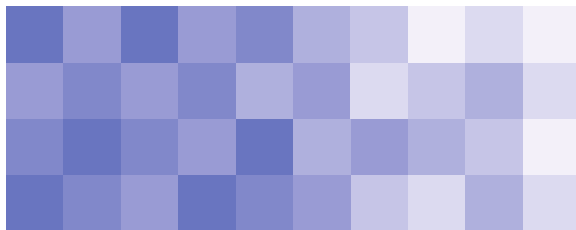

OUTSIDE LEGS
32 in x 23 in
(81 cm x 58 cm)


POWERED BY
drive

MFR # **146-12011KD-2**

ASSEMBLY INSTRUCTIONS


1. Remove contents from carton.
2. Lay transfer bench upside down on a flat surface.
3. Attach leg with the large suction cup (marked "A") by depressing the brass push pin and inserting into the receptacle with the matching "A". Repeat for second leg (Figure 1).

FIGURE 1

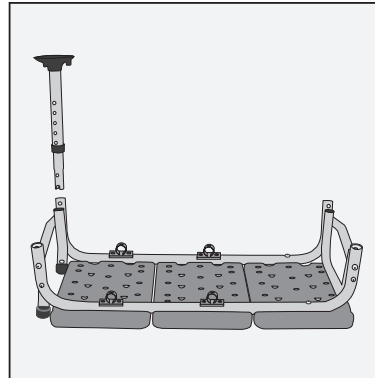


4. Attach leg with small rubber tip (marked "B") by depressing brass push pin and inserting into the receptacle with matching "B". Repeat for second leg (Figure 2).

FIGURE 2

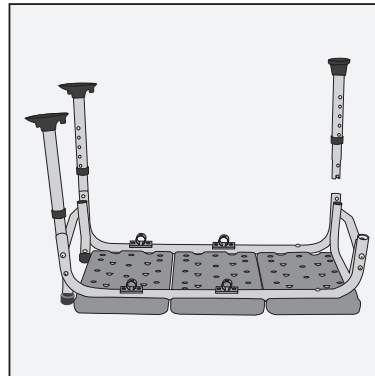


5. Turn transfer bench right-side-up. Attach armrest by depressing brass push pins and sliding into the arm receptacle on the side of the bench. Make sure the arm arcs outward, not inward (Figure 3).

FIGURE 3

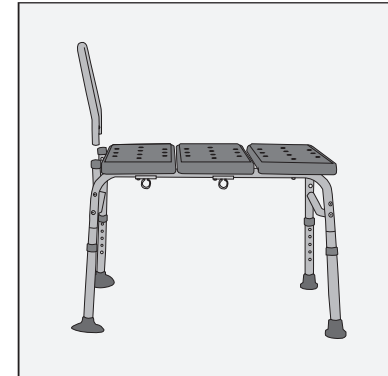

6. To assemble backrest, insert pole into backrest support by depressing push pin and sliding into receptacle. Repeat for second pole (Figure 4).

FIGURE 4

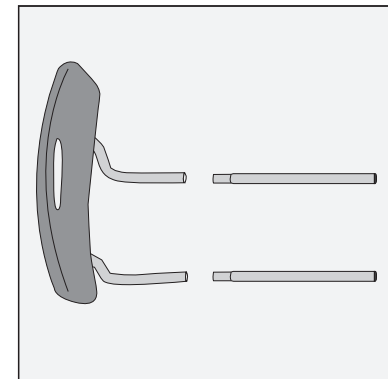


7. Insert backrest into base by depressing push pins and sliding supports through both receptacles.
Note: Backrest must go through both sets of receptacles to become secure (Figure 5).

FIGURE 5



NOTE: Back support is reversible to accommodate left or right tub entry. You can also change the seat depth of your transfer bench by pushing the pins under the seat and sliding the tubes forward until the pins lock into place.

8. Place legs with suction cups into bath tub.

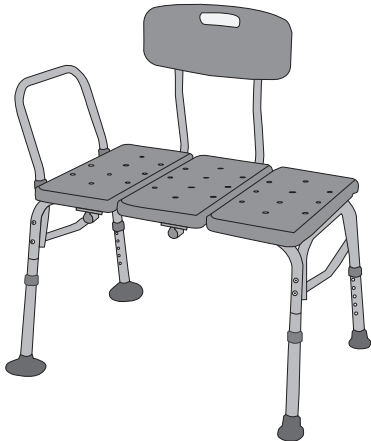

ADDITIONAL PRECAUTIONS

1. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Ensure ALL screws, nuts and/or bolts are tightened.
4. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
5. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
6. DO NOT stand on chair.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

ADJUSTING LEG HEIGHT

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
- NOTE: Make sure all legs are set to the same height.
2. To be sure legs are locked into place, push down or pull up on legs.

McKesson carries a comprehensive selection of durable medical equipment products for bath safety, mobility and aids to daily living, including wheelchairs, canes, bath benches, and more. Choose McKesson for quality, selection, and value.



Technical support, replacement parts and warranty information? Call 1-866-694-5085
General product questions? Call 1-800-777-4908

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by
McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN B0518
Made in China