




Handheld Shower Head with diverter valve

For use with MFR# 146-12037

INSTALLATION

1. Remove existing shower head.
2. Screw diverter valve into existing shower head.
3. Place rubber washer into diverter valve.
4. Screw diverter valve onto shower pipe.
5. Place rubber washer onto remaining hose nut.
6. Screw shower hose nut onto diverter valve.
7. Place rubber washer onto remaining hose nut.
8. Screw hand held shower head onto hose.
9. Mount bracket onto wall with screws using wall anchors.
10. Hang hand held shower head on bracket when not in use.

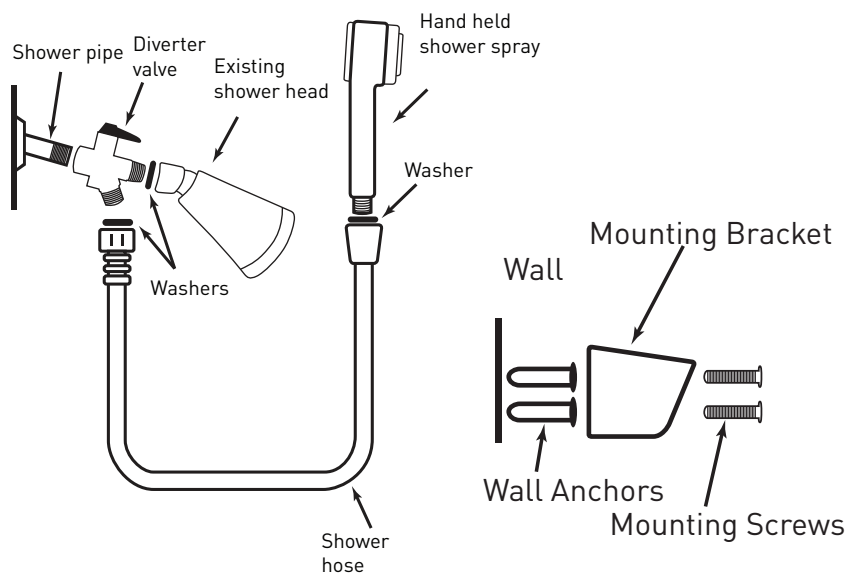

Technical support, replacement parts and warranty information? Call 1-866-694-5085

General product questions? Call 1-800-777-4908

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN B0618
Made in China