

Bath Bench


removable back | aluminum frame

Provides a total weight bearing capacity of 400 lbs (181 kg).


For use with MFR#'s 146-RTL12202KDR.

ASSEMBLY INSTRUCTIONS


1. Remove contents from carton.
2. Turn seat up-side-down. Place the leg section marked with a "B" into the groove on the bottom of the seat. Place leg section "A" on top of section "B" and secure both in place by inserting the shorter thumb screw through the middle hole as shown below.

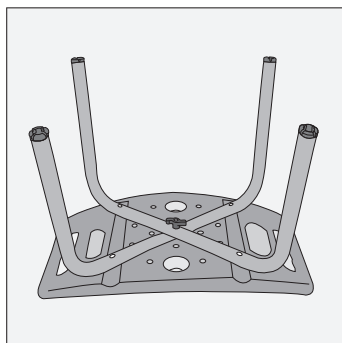

3. Assemble backrest by taking the "L" shaped backrest supports and securing them to the back rest using the shorter thumb screws as shown below.

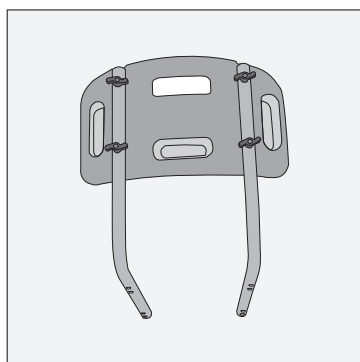


4. Attach backrest to the seat by aligning holes on "L" shaped frame supports with the holes on the leg sections. Secure in place using the longer thumb screws, making sure to pay attention to the "rear" of the seat marked on the bottom of the chair.

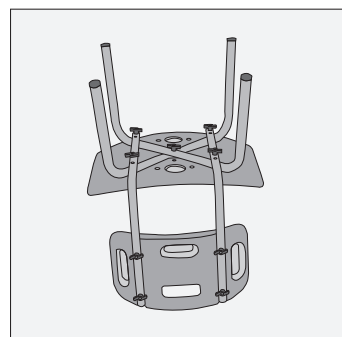

NOTE: Seat depth can be adjusted by choosing 1 of the 2 holes on the frame supports.

5. Insert legs into leg frames by depressing push pins, sliding legs over the receptacle and setting to desired height. Make sure all the legs are set to the same height.

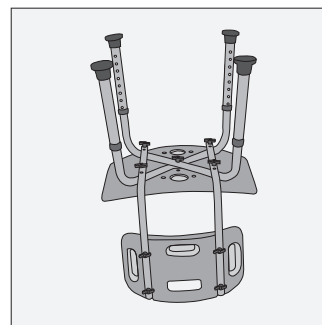

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration.
2. To be sure legs are locked into place by pushing down on or by pulling legs.

ADDITIONAL PRECAUTIONS

Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.

Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved. Ensure ALL thumb screws are tightened.

Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.

All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.

When used in a tub, make certain the rubber tips are not touching the sides of the tub wall.

DO NOT stand on chair.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN B0518

Made in China

