


McKESSON

Heel Protector

HOOK AND LOOP CLOSURES | TWO STRAPPING OPTIONS | SINGLE PATIENT USE

For use with MFR# 16-7305

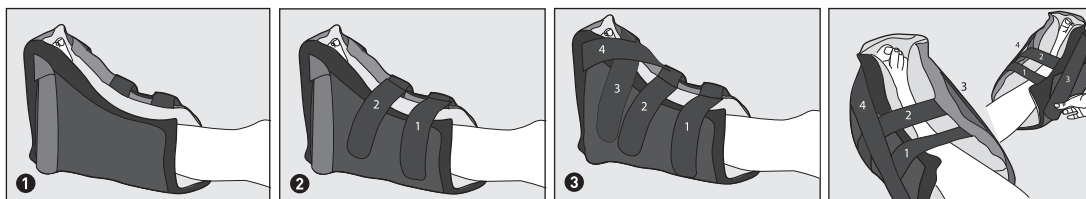

Option 1

Option 2

INSTRUCTIONS FOR USE

1. Unfasten each strap. Place the lower leg on the long side of the boot, with the heel positioned above the heel opening. Ensure heel is elevated.
2. Fasten strap 1 across the leg and secure on the opposite side. Repeat using Strap 2. DO NOT over tighten
3. Choose one of the following options to secure the foot portion of the boot:

Option 1: Cross strap 3 over the foot and secure on the opposite side near the ankle. Repeat with strap 4 to make an "X" pattern on the top of the foot.

Option 2: Apply added ankle support by diagonally stretching strap 3 from the toes down to the back of the leg without touching the skin. Repeat with Strap 4 on the opposite side.

CLEANING INSTRUCTIONS

For a light cleaning: Wipe clean with a damp cloth and allow heel protector to airdry.

For a thorough cleaning: Secure the straps to the heel protector and place in a mesh laundry bag. Machine wash in warm (100°F [38°C]) water. Tumble dry on low/delicate cycle.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN A0617
Made in China